

# IDL<sup>TM</sup> STANDS FOR IDEAL

/ Application steps for DDB / BOSTON: Stand and put on the orthopaedic brace with the opening to the back / Check that the middle of the orthopaedic brace's opening is in line with the middle of the back / Make sure it fits snugly around the pelvis / Check that the metal clips are inside the thermoplastic / Start adjusting the dressing straps through the buckles. Loosely tie the second bottom strap first, then the third, fourth and finally the first. Repeat the same process, tightening the straps more and more, up to the points indicated by the technician.

/ Application steps for CHENEAU / post-operative: Stand and put on the orthopaedic brace with the opening facing forward / Check that the middle of the orthopaedic brace's opening is in line with the middle of the body / Make sure it fits snugly around the pelvis / Start adjusting the dressing straps through the buckles tightening them up to the points indicated by the technician.

/ Always place the orthopaedic brace over a tight fitted, 100% cotton tank top.

/ It is necessary to observe the skin every time you take out the orthopaedic brace. Keep in mind that the skin at the pressure points (whiplash points) must adapt to the pressure it receives.

/ Take proper care of your orthopaedic brace, and maintain good hygiene of your body and of the orthopaedic brace. In the event of an allergic reaction or user discomfort or injury, please remove the orthopaedic brace and contact us as well as your physician.

/ The orthopaedic brace can be cleaned with soap and water or wet wipes or a wet towel - no harsh cleaners required.

/ The daily application schedule of the orthopaedic brace, as well as the total duration of the treatment, are determined by your physician.

/ The orthopedic brace should be checked by us 1-1.5 months after its first application. Afterwards and during the treatment, this check should ideally be done every 3-4 months and at least every 6 months without any extra charge. Do not neglect the periodic checks of the orthopedic brace. In case of any inconvenience, do not wait for the above period to pass, but contact us so that the necessary corrections can be made. The orthopedic brace should also be monitored by your physician at the time intervals she/he will determine.

/ Do not leave the orthopaedic brace in the sun, near radiators and heat sources or in the car especially during the summer months.

/ The orthopaedic brace is intended for use only by the person for whom it is manufactured.